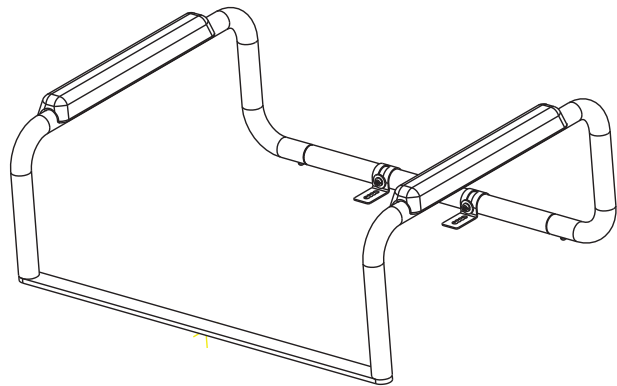
INNOVATIVE DESIGNS
for independent living™

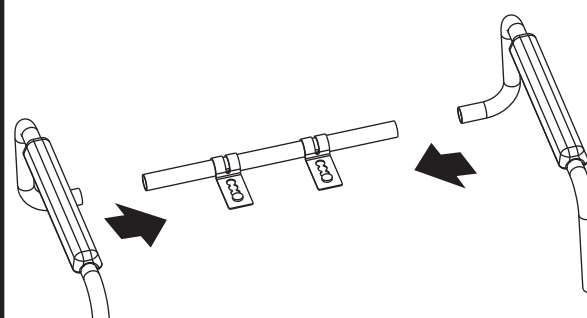
Tools Needed:
Herramientas necesarias:
Outils requis :

TOILET SAFETY RAILS / RIELES DE SEGURIDAD PARA INODOROS / BARRE D'APPUI POUR TOILETTE

1


Attach rails to center piece with brackets facing as shown.

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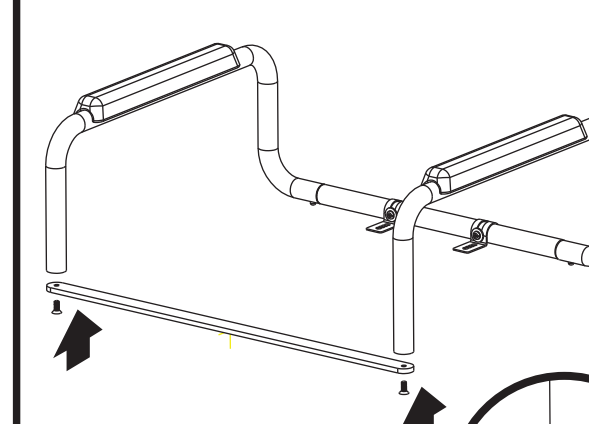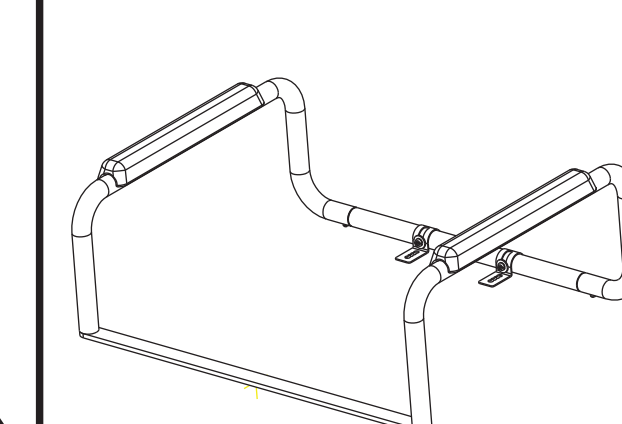
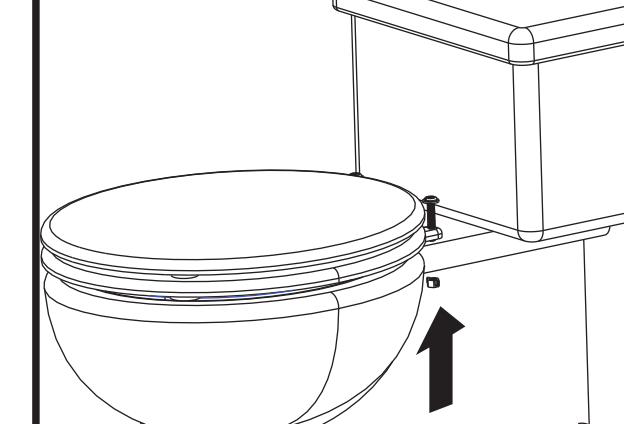
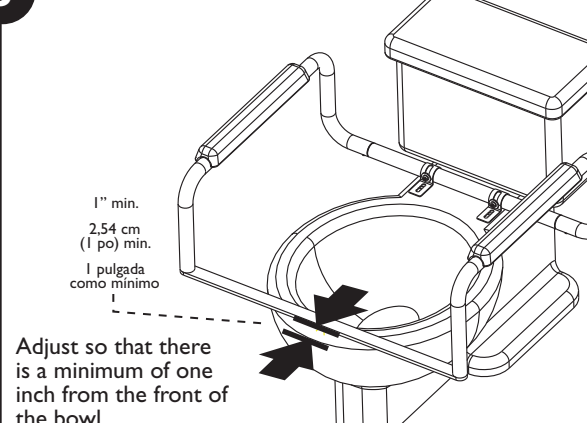
5

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
Adjust so that there is a minimum of one inch from the front of the bowl.

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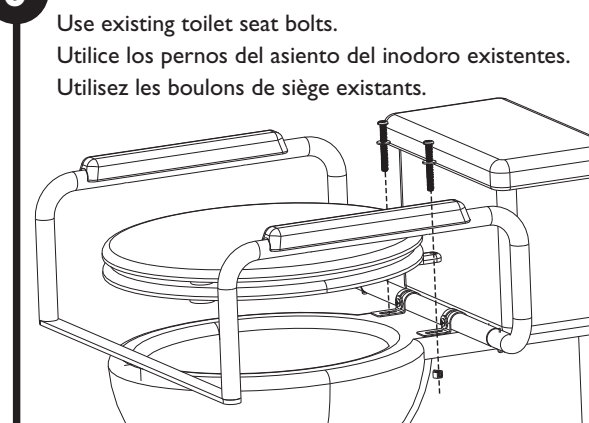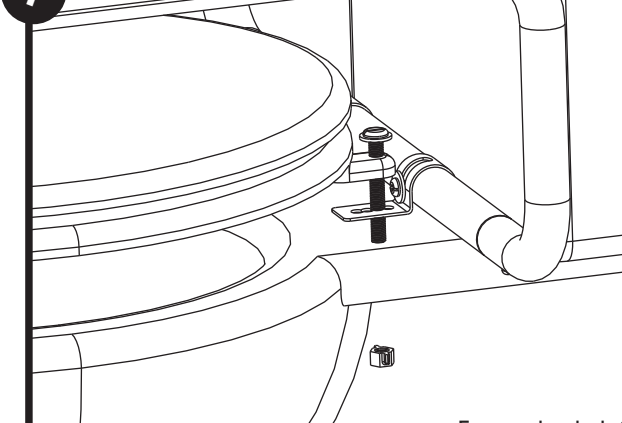
7


Ensure that bolt is securely fastened into the nut.

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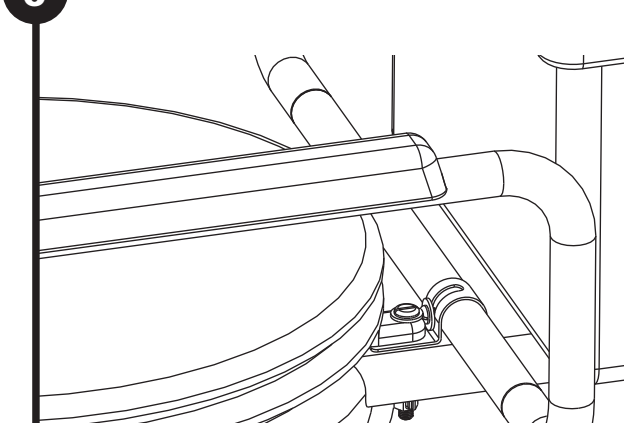8

Warning! Prevent Serious or Fatal Injury:

For use only by persons weighing up to 250lbs. Consult a health care professional for advice and instructions specific to your needs before use. Install in accordance with CSI's instructions. After installation, periodically check and maintain the condition of toilet seat bolts to ensure they are not damaged and that they clear the bottom of the toilet and securely fasten into the nut. If you have any concern about the length, condition or strength of the current toilet seat bolts and nuts, please replace them with an appropriate new set. Contact CSI immediately at 1-800-882-0116 if any components of the rail are damaged or missing, or if you have any questions concerning installation, use or maintenance of this product. Do not jump or stand on this product or modify it in any way.

FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION, CONTACT CSI AT 1-800-882-0116.

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