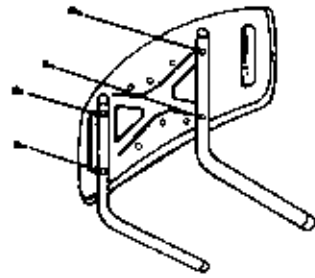
**Tools Required:**

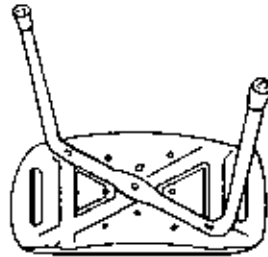
- Standard size Phillips head screwdriver

**Assembly Instructions****Step 1: Assemble Backrest**

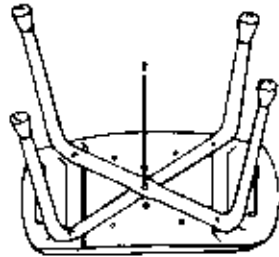
Place one L-shaped backrest support tube into the groove of the back of one of the white plastic seat panels so that the red arrow points up and is not covered. Attach tube to seat panel by inserting a 1½" screw through a washer and through the tube into each of the two holes on one side of the back of the seat panel. Repeat for the second L-shaped tube.

**Step 2: Position Legs**

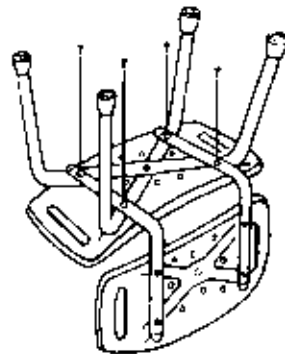
With second plastic seat panel upside-down, place leg "1" into groove "1" so that the number "1" marks are next to each other. Then place leg "2", over leg "1" so that the number "2" marks are next to each other.

**Step 3: Attach Legs**

Insert a 1½" screw through a washer into the center hole of the legs and seat. Tighten several turns.

**Step 4: Attach Backrest**

With seat still upside-down, place assembled backrest over seat assembly. Insert remaining four "2" screws through washers, backrest support tubes and legs into the bottom of the plastic seat panel. Tighten all five screws fully. Test seat. Retighten as needed.

**Step 5: Adjust Leg Height (DN7030)**

Push in button on leg, slide leg to desired height and allow button to pop out and lock leg in place. Repeat for each leg, making sure all four are adjusted to the same height.

**Cleaning instructions:**

Wipe clean with a soft, damp cloth. Detergents and cleansers may harm the finish of this product.

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FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION, CONTACT CSI AT 1-800-882-0116.

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www.homecare.moen.com

MOEN  
INNOVATIVE DESIGNS  
for independent living

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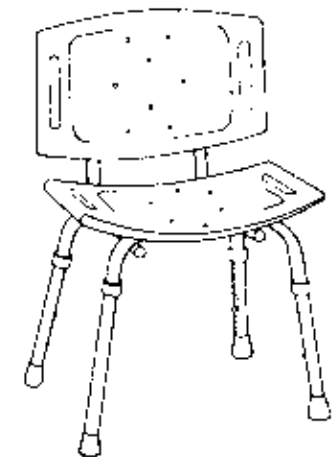
Dimensions: Adjustable height from 14" to 21"

20" wide x 12" deep x 16" high (seat)

Dimensiones: Altura ajustable de 14" a 21"

(355 a 533mm)

20" (505mm) de ancho x 12" (300mm) de

profundidad x 14" (356mm) de alto (asiento)

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(356 à 533mm)

20 po (505mm) de large x 12 po (300mm)

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hauts (siège)

### WARNING

This product has a maximum user weight of 250lbs. Do not jump or stand on this seat. Contact CSI immediately at 1-800-882-0116 if any components are damaged or missing. Making substitutions or modifying this product in any way will void your warranty. CSI assumes no liability for any damage or injury caused by improper assembly, installation or use of this product.

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Home Care is a registered trademark of Home Care Tim-BR-MART, licensed to Moen Incorporated. / Home Care es una marca registrada de Home Care Tim-BR-MART, cuyo uso fue autorizado a Moen Incorporated. / Home Care est une marque déposée de Home Care Tim-BR-Mart et Moen incorporée en est un utilisateur autorisé.