# Premium Transfer Bench Assembly Instructions

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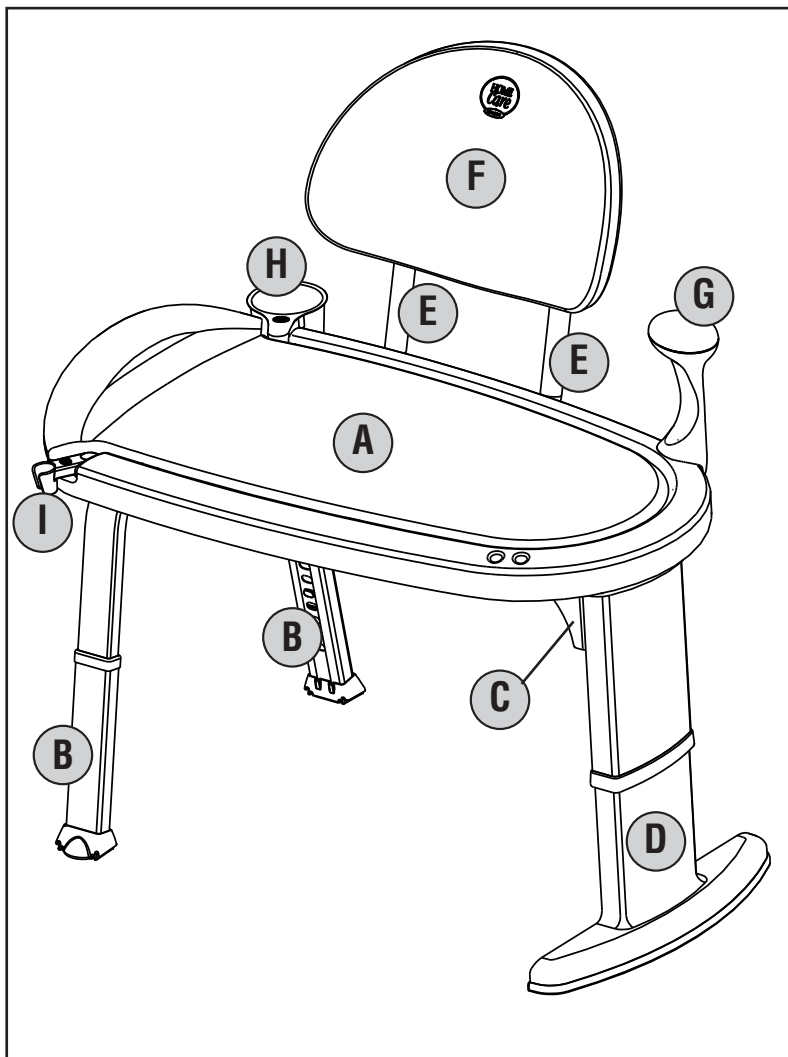
**WARNING: PLEASE FOLLOW THESE ASSEMBLY INSTRUCTIONS CAREFULLY. ASSEMBLER ASSUMES RESPONSIBILITY FOR PROPER ASSEMBLY. FAILURE TO FOLLOW CORRECT ASSEMBLY PROCEDURES CAN RESULT IN SERIOUS INJURY.**

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### List of parts

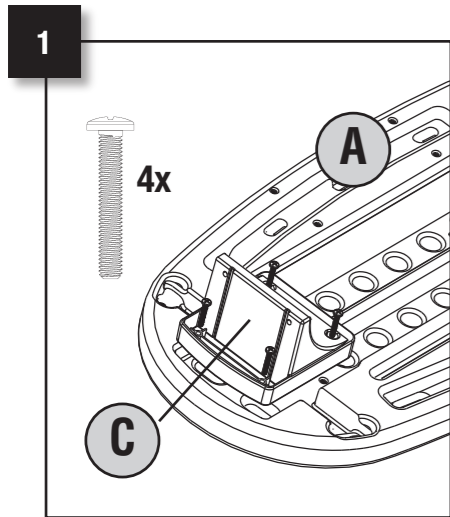
- A** Seat
- B** Rear legs with angled leg support
- C** Front support (ten 1 1/4" screws; two 5/8" screws)
- D** Front leg assembly
- E** Back support arms with back support screw package (four 1 1/2" screws; two 1/2" screws; four washers)
- F** Seat back
- G** Support handle ( two 1 1/4" screws)
- H** Basket organizer
- I** Shower holder

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**To assemble:**

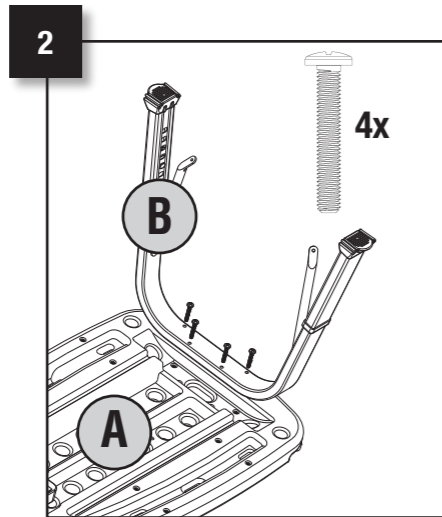
1. Turn seat (A) upside down. Attach front support (C) to front edge of seat using four 1/4" screws

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
2. Align screw holes, then attach rear legs (B) to back edge of seat (A) using four 1/4" screws.

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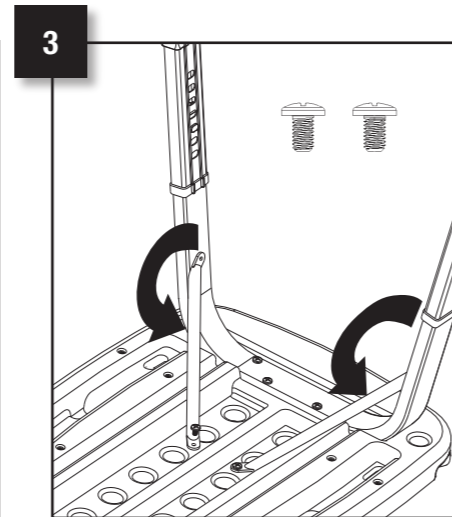
3. Rotate the angled leg supports (attached to rear legs) toward the front of the seat. Attach leg supports to seat using two 5/8" screws.

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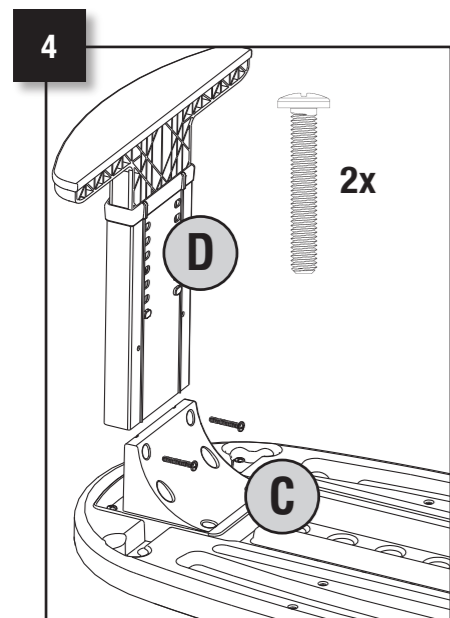
4. Insert leg assembly (D) into front support (C) and attach using two 1/4" screws.

4. Inserte la asamblea de las patas (D) en el soporte delantero (C) y sujétela con 2 tornillos de 1/4".

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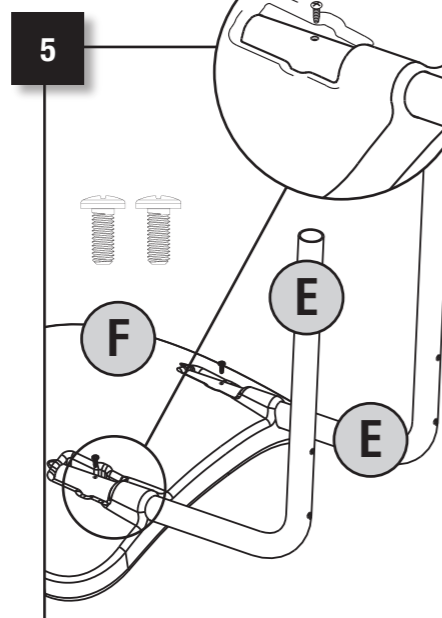
5. Insert the back support arm (E) into the hole located on rear of seat back (F). Insert a 1/2" screw into the small hole of the support arm (E), screw facing up.

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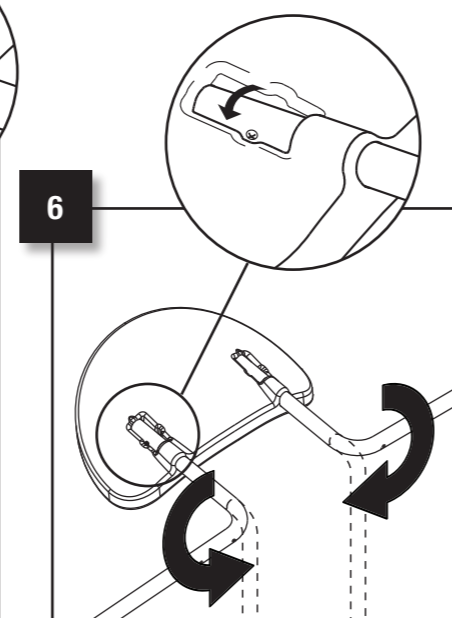
6. Align screw and slot, rotate back support arms 180° to lock the back support arm in place.

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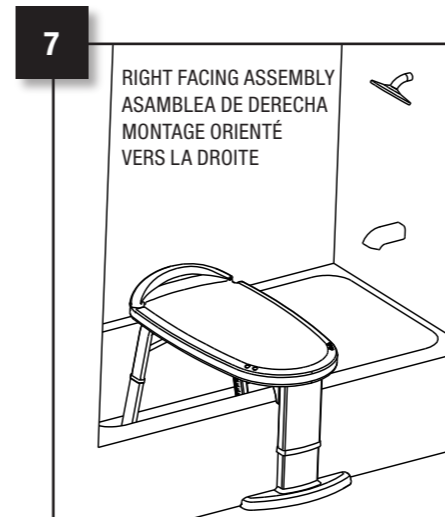
7. Transfer bench should be placed in tub to determine left or right seat and handle assembly.

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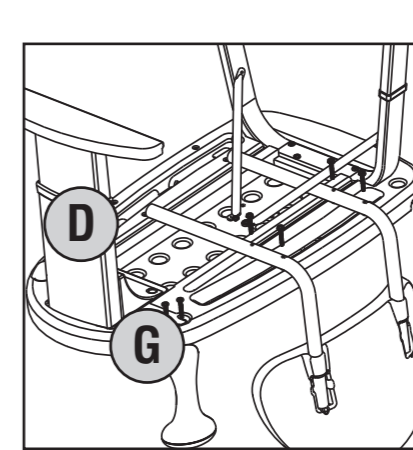**Right facing assembly**

- a. Place seat upside down, with front leg assembly (D) closest to you. Attach legs of seat back on the **right** side of the seat, using four 1 1/2" screws and four washers.

- b. On the same side, attach support handle (G) between the front leg assembly and seat back using two 1/4" screws.

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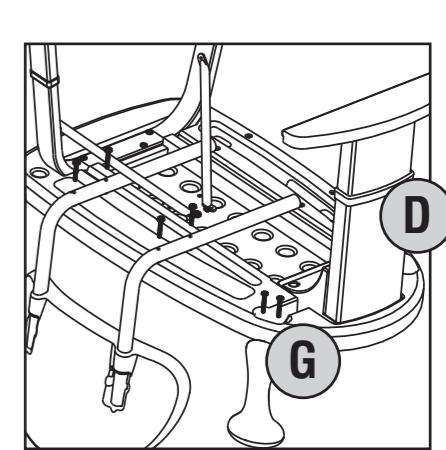
**Left facing assembly**

- a. Place seat upside down with front leg assembly (D) closest to you. Attach legs of seat back on the **left** side of the seat, using four 1 1/2" screws and four washers.

- b. On the same side, attach support handle (G) between the front leg assembly and seat back using two 1/4" screws.

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
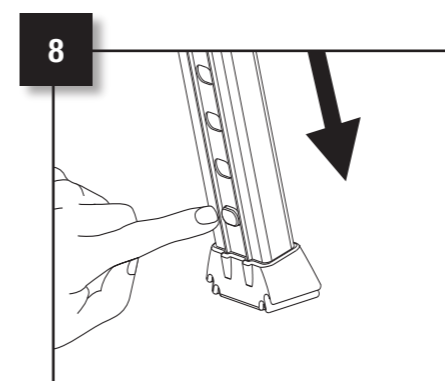
8. Adjust transfer bench to the appropriate height. Adjustment buttons located on inside of the legs.

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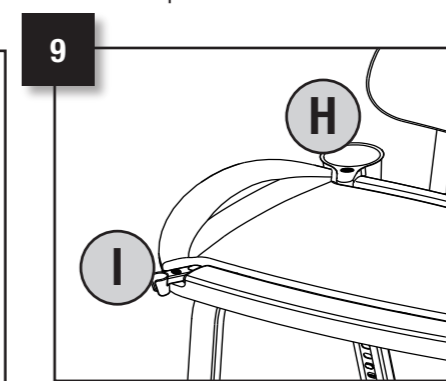9. Insert basket organizer (H) and shower holder (I) as shown.

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INNOVATIVE DESIGNS  
for independent living™

**WARNING:**

TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 400 POUNDS. Consult a healthcare professional for advice and instructions specific to your needs before use. Contact CSI immediately at 1-800-289-6636 if any components are missing or damaged. Install in accordance with CSI's instructions. Do not modify this product in any way. FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT CSI AT 1-800-289-6636.

**LIMITED LIFETIME WARRANTY:**

If this product fails due to materials or workmanship at any time during the life of the product, Creative Specialties International (CSI), a division of Moen Incorporated, will replace it free of charge, postage-paid. Simply contact CSI at 1-800-289-6636 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn.

TO THE EXTENT PERMITTED BY LAW, CSI DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

**LIMITATION OF LIABILITY:** THIS WARRANTY IS YOUR SOLE REMEDY AND CSI SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL CSI'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit [www.homecare.moen.com](http://www.homecare.moen.com)

Home Care by Moen Premium Shower Chair meets or exceeds the applicable requirements of the following standards/guidelines:

 The Americans with Disabilities Act ( ADA )

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Creative Specialties is a division of Moen Incorporated. © 2008
Home Care is a registered trademark of Home Care Tim-BR-MART, licensed to Moen Incorporated.
Home Care and Creative Specialties are a registered trademarks of Moen Incorporated.